

NUTRICIA floca⁽re[®] Infinity[®] III **GO-FRAME INSTRUCTIONS FOR USE** SETTING UP THE GO-FRAME




Although the shelf is moveable, all bottle sizes fit the lowest position.

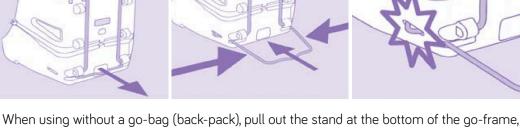

until it clicks into position, to make it more stable.

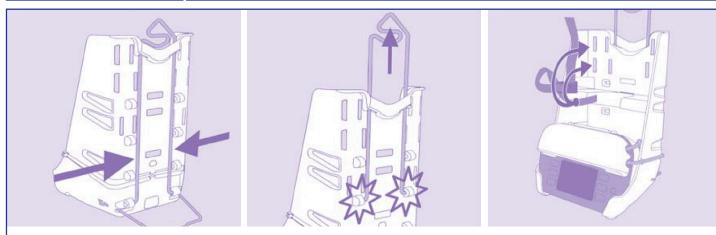

The stand at the back can be moved up for 1000ml OpTri bottles (ensure it clicks into place), remember to also move the straps up to the same level. For all other bottle sizes, it should remain at the 'default' lowest position.

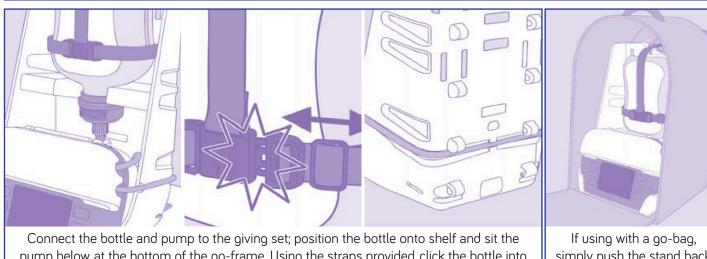

pump below, at the bottom of the go-frame. Using the straps provided, click the bottle into position, then wrap the giving set into position.

simply push the stand back in and place into go-bag.